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**OPERATING INSTRUCTIONS**

ECO-Finisher Macerator  
750-E1.3

---

# Inhalt

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## Index

Basic information		23
Paper containers used in device		24
Installation	Installation	25
	Water inlet	25
	Connecting to waste	25
	Electrical installation	26
	Establishment of device	26
Device location		27
Control and signaling elements		28
Manipulation	Manipulation	32
	Use of the device	32
	Disinfection refilling	33
	Change in volume of the disinfection container	33
	Autostart	34
Trouble shooting	Emergency lid opening	34
	Emergency drain	35
Safety instructions		35
Cleaning and maintenance	Cleaning	36
	Maintenance	36
Technical Specification		37
Dimension drawing		38

Attention

**Before the device use or its putting into operation, please, unconditionally read the manipulation book and observe safety instructions**

## Conformity Declaration

The manufacturer declares that under the Act No.22/1997 as amended by subsequent legislation issued on this product "Conformity Declaration".

## Basic information

Device EKO-Finisher is used for fast, hygienic and cost effective ecological disposal of paper disposable sanitary containers, including excrement and toilet paper. Device is equipped with a new system of opening the cover of the crushing container and with multistep automatic disinfection.

The new disposable hygienic containers – bedpans, kidney bowls, sputum cups, urinal bottles etc. - are made from recycled paper and are fully biodegradable and are frayed by supplying a certain amount of water / liquefied in the machine and then released into the sewage. Device may scrap at one time a mixture of different paper bins. The entire automatic process of crushing takes 3 minutes or longer, depending on the selected type of disinfection.

Liquidation system is primarily intended for use in social and health care that care for people with limited mobility. The optimum locations for this new system, all agencies and institutions the daily patient aged and elderly persons in care offer a high degree of hygienic safety. Transmission of nosocomial infections by this system is almost impossible. This system allows significant savings in operating costs compared to the existing method - washing dishes and is neither time-consuming operation.

### Attention

The device can be disposed of only disposable containers, recommended by the manufacturer and the retailer. The device is not intended for disposal of rubber gloves, syringes, paper diapers, incontinence pads, bandages or metal objects, etc.

If faults occur in the device due to unauthorized disposal of materials and articles shall not apply to these deficiencies agreed warranty!

## Paper containers used in device

All sanitary containers are used only once, thus avoiding the possibility of cross infection. The advantage of sanitary disposable paper containers is that the inner surface is made of a special molding and remains at least 24 hours watertight. An exception is the urine container which is watertight only 6 hours. The containers are then including the content of the device within 3 minutes completely crushed and discharged to waste.

Examples of the maximum number of paper containers per cycle:

<b>Toilet receiver bowl PM-2P</b>		3 pcs
<b>Bedpan PM-3</b>		3 pcs
<b>Bedpan PM-4</b>		3 pcs
<b>Urinal bottle UB-1</b>		4 pcs
<b>Kidney bowl EM-1</b>		8 pcs

## Installation

Installation of this device can only be carried out by a qualified person or a producer's / seller's service mechanic.

The device must be installed in the suitable room where the necessary hygienic conditions are secured and where the following necessities are provided:

- power supply 230V / 50Hz - a socket
- water supply equipped with the closing valve 3/4"
- drain connection of nominal ID 70 ( or nominal ID 100 with reduction )
- flat and waterproof floor so that the device could be installed in the absolutely horizontal way. Plastic foos with the nonskid rubber base are adjustable by the screw thread. The device can be at customer's request anchored with the screws to the floor.

The device should only be installed in the rooms that are enough sound insulated so that the noise level in the surrounding rooms does not exceed max. 30 dB (A).

When the device is installed near the wall, arrange enough space for the side cover dismantling (see pic. A, B, page 26)! The minimum dimensions are given, a bigger space is recommended for the easier manipulation.

### Water inlet

Rubber hose that on both ends 3/4 "connection fitting is Included in the delivery device. Connection to the inlet valve is placed on the back bottom in right corner of the device.

Rubber hose and the inlet valve are provided with a filter sieve preventing the penetration of dirt into the machine.

For water supply is used only cold water. To connect to a wall is recommended angle valve with 3/4" thread. Water inlet should be preferably installed in the immediate vicinity of the device, but preferably directly behind him so that the manual is accessible even after the installation.

### Connecting to waste

The rear drain unit has a nominal diameter of DN 70. Use a suitable adapter if the waste pipe is larger diameter.

Connection to the fixed waste must not be placed higher than the drain from the device. Waste should be made out of plastic pipe.

During the operation of this device is imperative that the drain pipe has been vented (eg. Ventilated - flue air on the roof).

Attention

The connection to the drain with the smaller nominal diameter than DN 70 is not recommended. It is recommended to solve the problem with the manufacturer or importer.

### **Electrical installation**

Connecting to the mains socket must be done in a clearly visible and freely accessible place in the immediate vicinity of the device. Circuit for device connection must be separately protected by a fuse or circuit breaker 16 A and in compliance with the provisions of the relevant standards and other regulations and in particular the following requirements:

- Electrical installations must be performed organization that is authorized to work on electrical equipment.
- In case of doubt about the satisfactory and safe electrical installation must be done before connecting the initial inspection.

### **Attention**

If the socket is not freely accessible and visible, well visible and accessible closing main switch must be installed (pic. B, page 27).

A damaged power supply shall be replaced for the new one by the manufacturer or the professional service.

The electrical installation must be carried out by an organization authorized to work on electrical equipment in accordance with technical standards.

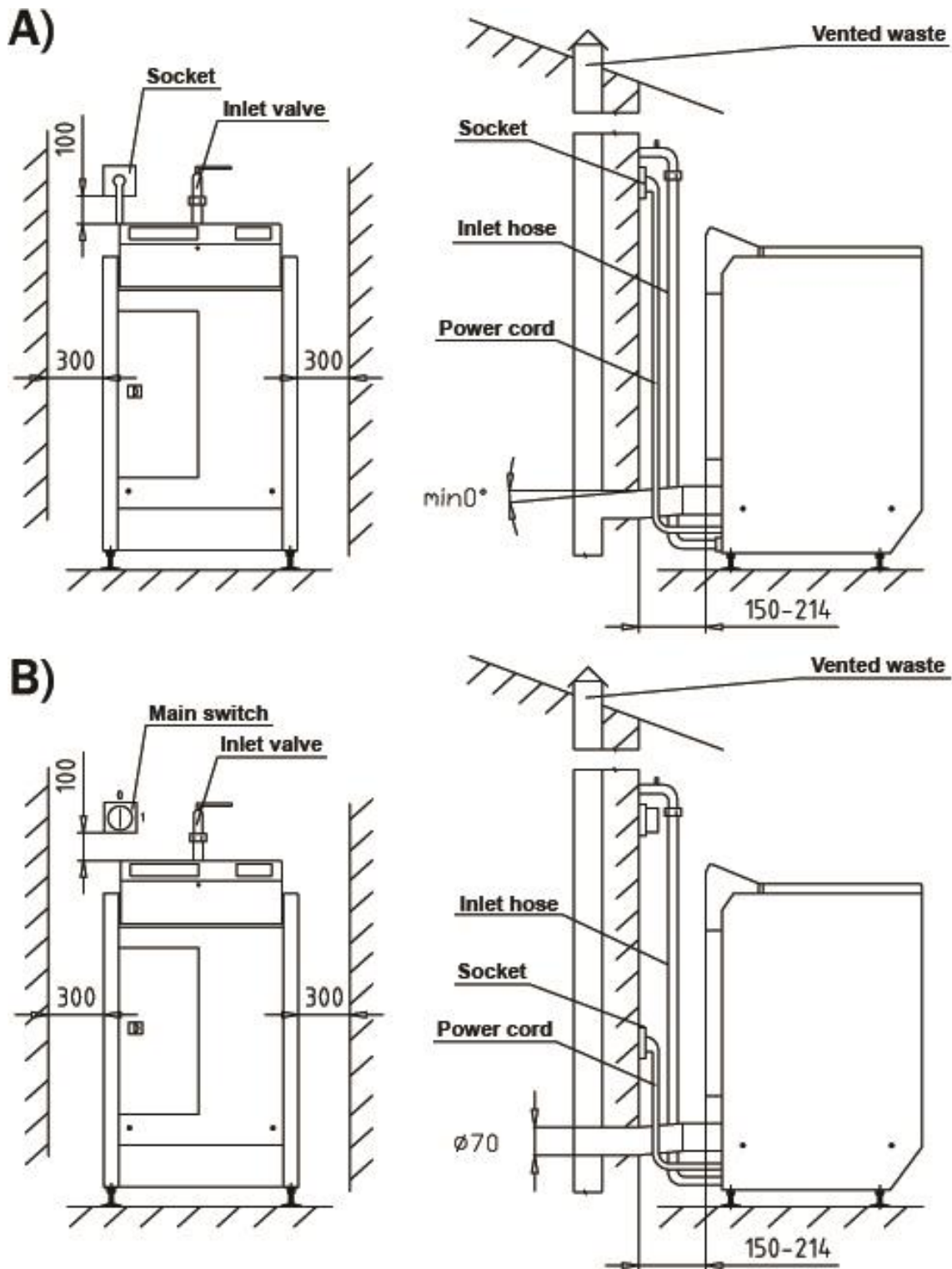
In case of doubt about the suitable and safe conduct of the electrical installation, an initial inspection must be carried out before connecting.

### **Establishment of device**

The device is established by 4 adjustable feet with rubber sleeves. Place the sleeves on the feet before the establishment of the device. Set the machine to horizontal position by unscrewing the feet.


The device can be equipped with a pedestal for places with a high drain outlet. Also the device can be equipped by pedestal with wheels for easier manipulation.

## Device location

These dimensions are the minimum, it is recommended for larger easier manipulation.

## Control and signaling elements
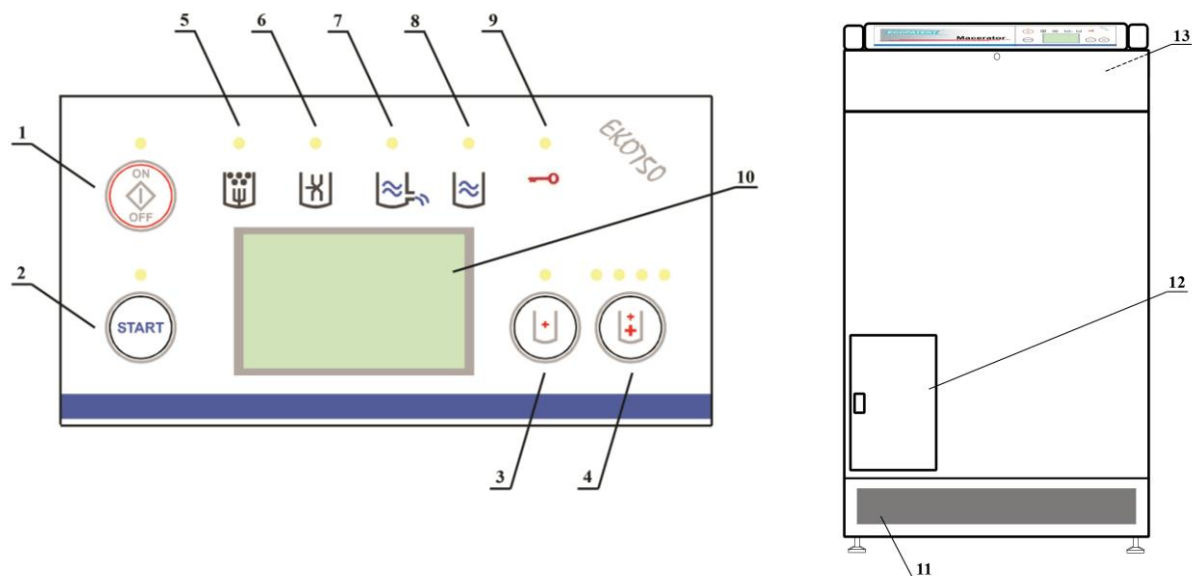
### 1) Button ON/OFF

Button ON/OFF button switches on and off control circuit of device.

- OFF – the light is red, the device is switched off and is connected to the electricity network
- ON – the light is green, the device is switched on
- the light flashes red / green – device failure

### 2) Button START

START button starts a 3-minute duty cycle.

- the light is red – system is ready
- the light is green – working cycle is running
- the light flashes red / green – pause in the cycle, cycle interruption due to power failure

If during the cycle again press the START button, the working cycle pauses, flashes red / green and the lid can be opened. However, the system evaluates the lid opening as a cover error. You can continue to press again START. The light also flashes red / green when the cycle was interrupted due to a power failure. Press the button „START“ to continue in the working cycle.

Working cycle can also be started in the absence of water in the tank. Display shows "PROCES STARTING", EKO-Finisher wait until the water is filled in the tank. Then automatically starts the crushing cycle.

If the autostart function is set, the START button can be used to cancel the automatic start during the entire countdown (10 seconds).

### 3) Button standard and extended disinfection

- the light is yellow – disinfection deactivated
- the light is green – standard disinfection activated
- the light flashes green – extended disinfection activated

Standard disinfection is activated by pressing the button (the light is green). It means that rinse of the tub with water and disinfection is at the end of working cycle.

Extended disinfection is activated by second pressing the button (the light flashes green). It means that rinse of the tub with water and disinfection is at the end of working cycle but in addition after this rinse water with disinfection stay in the tank for one minute and the device is closed for 1 minute and cannot be opened at this time. By selecting the extended disinfection extends the cycle time to 4 minutes.

Standard disinfection or extended disinfection can be activated before starting the cycle or at least 1 minute and 30 seconds after the start of the cycle.

Disinfection is deactivated by third pressing of the button.

Selected disinfection function is active only for one operation cycle.

The final concentration of the disinfectant solution to disinfect the tub interior is approximately 1%.

### 4) Button active disinfection

Active disinfection is used for disinfect the content during crushing. There are three levels of setting which differs in the length of treatment. LED indicators inform about length of treatment.

- the light is yellow – disinfection deactivated
- 1 light is green – activated active disinfection – treatment 5 minutes
- 2 lights are green – activated active disinfection – treatment 10 minutes
- 3 lights are green – activated active disinfection – treatment 15 minutes

Disinfection is filled into the grind content and allowed to act for the selected length of treatment. Disinfection with length of treatment 5 minutes is activated by pressing the button (1 LED indicator is green). Disinfection with length of treatment 10 minutes is activated by second pressing the button (2 LED indicator are green). Disinfection with length of treatment 15 minutes is activated by third pressing the button (3 LED indicator is green). Disinfection is deactivated fourth by pressing the button. After the selected period of active disinfection cycle goes standardly on next steps.

Active disinfection can be combined with standard or extended disinfection.

Selected disinfection function is active only for one operation cycle.

Active disinfection can be activated before starting the cycle or at least 1 minute and 30 seconds after the start of the cycle.

### 5) Indicator blockade motor / pump

In case of blockage of the pump or motor is interrupted operation cycle. The light flashes red and on the display is shown error message "SYSTEM ERROR - motor blockade" or "SYSTEM ERROR – pump blockade". The lid can be opened.

The most common cause of motor block is jam of paper containers in the drum or attempt to crushing inappropriate contents (bandages, plastic, molded pulp cloths etc.).

In case of this fault the EKO-Finisher must be turned off, jammed container released or foreign objects removed. In the case that the program is stopped at the time when filling is already crushed, it is possible to use the "Emergency drain" (page 35) for early drain the filling. After removing the cause, it is possible to close the lid again and continue the program by pressing the button „START“.

#### 6) Indicator drain valve

The light shines if the drain valve is not possible to open or close. Operating cycle is interrupted and the light flashes red. Error message "SYSTEM ERROR – blocked drain" is shown on the display.

See the waste of the machine if an error occurs. Look, if in the waste of the machine isn't a foreign object defend closing the discharge. If necessary, remove the object. Contact service if the fault recurs.

#### 7) Indicator liquid leak

The indicator signals together with the intermittent sound signal liquid leak (water, disinfection) into lower safety tub – device turned off automatically. Error message "SYSTEM ERROR – leakage water" is shown on the display. In the case of water leak close the water supply, disconnect the device from electrical power and contact service.

#### 8) Indicator water volume

This light indicates the volume of the water in the water tank. The operating cycle cannot start if the device does not have enough water. Filling starts if necessary after connecting the device to the power supply, closing the lid and the machine is turned by the button „ON / OFF“.

- the light is blue – enough water in the tank, device ready to start
- the light flashes blue – lack of water in the tank, the water is refiling

The system evaluates the problem if it is not filled with the water within minutes. Error message "LACK OF WATER – not fill" is shown on the display. It is necessary to check the water supply.

#### 9) Indicator lid opening

The light flashes red and error message "COVER ERROR – wrong closed" is shown on the display if you press the switch for opening the lid and the lid cannot be opened.

In the case of error try to cover "close" by pressing and open it again by pressing the button or key placed in a holder in the chamber disinfection. Description of emergency opening, see page 34.

## 10) Information display

EKO-Finisher is equipped with a large and clear display, which:

- provides information about the cycle, shows the time left to the end of the cycle
- indicates the status of the machine
- displays error messages

Overview of the information display:

- |                                  |                                    |
|----------------------------------|------------------------------------|
| a) SYSTEM READY                  | i) SYSTEM ERROR - leakage water    |
| b) SYSTEM READY - cover opened   | j) SYSTEM ERROR - blocked drain    |
| c) PROCES STARTING               | k) COVER ERROR - wrong closed      |
| d) PROCES RUNNING                | l) INTERNAL ERROR - communication  |
| e) COVER ERROR – cannot open     | m) LACK OF WATER - not fill        |
| f) SYSTEM NOT READY              | n) FATAL ERROR - pull out the plug |
| g) SYSTEM ERROR - motor blockade | o) CHECK DESINFECTIOIN             |
| h) SYSTEM ERROR - pump blockade  | p) RESET TO ZERO                   |
|                                  | q) AUTOSTART                       |

## 11) Foot opening

Lid opening is controlled by lightly pressing the control surfaces of the bottom of the machine by feet for about 2 seconds. The lid is then automatically opens to the maximum position.

This function can be used only when the machine is switched on (ON / OFF button is green), but not during running operation cycle, when the lid is locked. In case of power failure or interruption of electrical power, use the "Emergency opening" (page 34).

The lid is closed manually by the operating staff, pressure on the upper part. The lid must be compressed until it is reliably closed. The lid is secured against opening and automatic opening is eliminated during the running of the operating cycle. If it is necessary to open the lid during the cycle, it is first need to interrupt the operating cycle (press START botton). Incorrect closing or opening of the lid during interrupted cycle, the error message "COVER ERROR - wrong closed" is shown on the display.

## 12) Door of disinfection

Container for disinfection and a key (T-shaped) for emergency opening of the lid are placed behind the door of disinfection.

Machine is equipped with disinfection container. Is it possible to equip the device with another suitable container (for 2, 3, 4 or 5 liters).

## 13) Fuses

Instrumentation fuses in pouches serve to secure components. They are located on the back of the panel. If necessary the fuses can be replaced after unscrewing lids. Replacement of fuses can be performed only by an authorized person.

## Manipulation

Having read the manual resp. having acquainted with it, the manipulation of EKO-finisher is very easy. In spite of that the manipulating staff must keep basic safety precautions, especially use protecting rubber gloves, for their own safety - danger of the infection while handling the human waste.

Own operation of the EKO-Finisher is very simple after familiarization with this manual. Operating staff must observe safety regulations for working with fecal waste (gloves etc.). During normal use take care to avoid placing your hands in the drum of device. It is necessary to take extra care in order to avoid dangerous contact with sharp knives at the bottom of the tub if it is necessary to manipulate with already embedded paper containers.

## Use of the device

### Attention

You can put the paper container with excrement, toilet paper or paper towels, trace amount of pulp, water or urine in the device.

Any other objects could be liquidate in the device and inserting of any other objects is prohibited. These include plastics, metals, bandages, and wipes from molded pulp or nonwoven fabric.

- Plug in the mains plug (if it is not) and provide the power supply and open the water supply.
- Press the button ON/OFF (page 28, position 1). Device is turned on and the light is green.
- Open the lid by foot opening (page 28, position 11).
- Make sure that no foreign objects are in the tub. If so, unplug the device from the mains, remove the foreign object and repeat working cycle from the beginning.
- Insert the paper containers for disposal. Don't exceed the maximum number of containers (page 24).
- Close the lid by the pressure on the upper part. Push the lid until it latches securely into lock.
- Select the desired disinfection level and active disinfection
- Press START (page 28, position 2) to start the working cycle. Start of a cycle is signaled by green light and on the display is shown „SYSTEM READY” or „PROCES STARTING”. If a fault occurs, the cycle stops and a message about the defect is shown on the display.
- After the end of the cycle, the machine can be opened again and filled with the containers again or turn it off using the button ON/OFF.
- Working cycle is started automatically after closing the lid, if the function AUTOSTART is setting. Is it possible to interrupt working cycle by pressing button START (10 seconds after start).

The working cycle can be started before filling water into the device. In this case, the lid is locked and the machine will start the selected program after filling with water. The system evaluates the problem if it is not filled with the water within 4 minutes. Error message "LACK OF WATER – not fill" is shown on the display.

In the event that the device will not use a long time, close the water supply and disconnect the power supply from the wall socket.

### Attention

- All the objects inadmissible for the disposal must be removed from the tub before putting the device into operation. Before their removing unconditionally switch off the power supply or pull out the plug from the mains!
- If operation of the device is interrupted due to the fallout loss of electrical energy, finish the operation cycle by pressing the push-button „START“ after the supply is restored.
- Interrupting the cycle is characterized by flashing the start button (red / green).

## Disinfection refilling

The machine is supplied with disinfection container. The machine signals outgoing disinfection from 15 doses until the end. If the program with disinfection is selected, 3 seconds acoustic signal sounds after pressing the button „START“ and message “CHECK DESINFECTIION“ is shown on the display.

### Attention

When disinfection is activated after the start of the program, device doesn't notify outgoing disinfection.

Container for disinfection is placed behind the door of disinfection (page 28, position 12). After opening the container unscrew the cup and remove the suction hose. Do not remove the cap from the tube. Remove the container and fill it with disinfection. In the handling of disinfection be careful on personal safety. Full container with disinfection insert into the chamber and insert the suction hose. Screw in cap and push the suction hose into the container so that it reached to the bottom of the container.

### Attention

Do not put a container which is externally contaminated. Do not put a container that is damaged. Avoid spilling disinfection.

You have to reset dose counter after adding disinfection.

Resetting: the device is switched on (light ON/OFF is green), press together 2 buttons for disinfection (page 28, positions 3 and 4) for 3 seconds. On the display „DELETED“ is shown.

## Change in volume of the disinfection container

EKO-Finisher allows use of containers for disinfection of a 2, 3, 4 or 5 liters volume. The manufacturer supplies disinfection container and volume of the disinfection tank is setted in menu. It is necessary to change the container volume if you want to use a container with other volume. The device will not correctly signal the lack of disinfection without setting of right volume.

- To change the volume of the container disinfection is necessary to enter the service menu.
- The device has to be switched off. Turn off the device by pressing the ON / OFF  button. The light is red.
- Press and hold 2 buttons - start and standard disinfection.  
- Repeatedly change pages of menu by button standard disinfection  until sign „Disinfection“ is shown on the display.
- Number shows the actual volume of the container, 2 (for container 2 liters volume), 3 (for container 3 liters volume) or 5 (for container 5 liters volume). The bottom line of the display shows the number of cycles of disinfection used since the last reset.
- Press the button extended disinfection for change settings of the container volume disinfection. 
- Press the button ON/OFF for exit service menu .

## Autostart

The autostart function allows automatic start of the working cycle after the machine is closed. This function could be set in service menu.

### Attention

The autostart function is only activated if the lid is open and closed when the machine is switched on (ON/OFF lights green).  
Function Autostart will not start if there is an error.  
Function Autostart is disabled if it is not set in the settings.

If the autostart function is set, the START button can be used to cancel the automatic start during the entire countdown (10 seconds).

- Is it necessary to enter the service menu to change setting of Autostart.
- The device has to be switched off. Turn off the device by pressing the ON / OFF button. The light is red. 
- Press and hold 2 buttons - start and standard disinfection.  
- Repeatedly change pages of menu by button standard disinfection  until sign „Autostart“ is shown on the display.
- Press the button extended disinfection  for change settings of the Autostart function.  
3 settings: 1) disabled  
2) from start: yes  
3) from start: yes  
from pause: yes
- Press the button ON/OFF for exit service menu .

## Trouble shooting

### Emergency lid opening

If the device cannot be opened by foot it is possible to use key (T-shaped) for emergency opening. Key is placed behind the door of disinfection (page 27, position 12) in holder for tank disinfection.

Key inserts an angle of approx. 20° to the hole in the front side of the lid. About two-thirds length of the mandrel runs into the hook lock. You have to mild circle with key and you find dinge in the hook. Insert mandrel of the key in the dinge. Press by hand and the lid will open.

### Attention

If it is not possible to move with hook, it is possible, that lid is still closed. Don't try to open the lid by force. Press down the lid as you would like it to close and try to press the key again. If you still cannot open the lid, contact your service

## Emergency drain

Emergency draining is used when it is necessary to drain the contents of the tub before ordinary draining in the automatic program. For emergency drain have to be the device connected to the mains and fuses have to be ok.

- To emergency drain is necessary to enter the service menu

- The device has to be switched off. Turn off the device by pressing the ON / OFF button. The light is red.

- Press and hold 2 buttons - start and standard disinfection.

- Report „open drain“ will appear on the display.

- Start emergency drain by pressing the button active disinfection.

- Press the button ON/OFF for exit service menu

## Safety instruction

No admittance to unauthorized persons during the device operation.

Check regularly whether the mains are not damaged: **water supply - draining – power supply.**

Whether abnormal operation sounds or defects occur or if you have a feeling that the device does not work properly, switch it off immediately (button ON/OFF), do not further use the device and inform your service organization.

If you will hear shortly after starting the device (up to 5 s), or even during operation continuous beeping, the machine stops, the red indicator light flashes (page 27, position 6) and the display will show an error message "SYSTEM ERROR - blocked drain" there is a reasonable suspicion that in the drain valve remained dirt (remnants containers, pulp, etc.).

To resume normal operation turn the machine off, pull out the mains plug, remove dirt and start again. Press the button „START“. Contact your service if you will hear a beep again.

### Within the scope of the defect searching, check first:

- any foreign objects have to be found in the tub,
- all the supplies and outlets intact,
- it is not possible to open the lid,
- overheating, smell of smoke.

## Cleaning and maintenance

### Attention

By the cleaning and the maintenance please respect the safety regulations from this manual, especially the informations at the page 23 and safety instructions at the page 35.

### Cleaning

Make a cleaning of the tub by starting working cycle with disinfectant.

**DO NOT CLEAN THE INSIDE OF THE WORKING TUB MANUALLY. THERE IS RISK OF INJURY!**

If the inside of the tub and/or bottom of the lid is extremely dirty, make sure that cover of washer (black cover) is not damage. If so, contact your service.

**UNPLUG THE DEVICE FROM THE MAINS BEFORE CLEANING THE EXTERNAL PARTS AND AROUND OF THE WORKING TUB.**

We recommend keeping the device clean. Clean the outer surfaces with closed cover with non-abrasive detergent and disinfection with a soft cloth. Do not use solvents. Open the lid and clean working place around the tub and seals. Dry dirt first moistens with water. If needed also clean edge of white plate under the lid.

### Maintenance

- If necessary, fill the disinfection reservoir with the not diluted detergent.
- Check the water supply and electricity with respect to tightness and entireness!
- Check, if the black cover of washer under the lid didn't fall down.
- Check, if gasket or white plate under the lid is not broken.
- If you will find some defect, you have to switch off the device set out of operation and contact service.
- In case of long term of set out of operation clean the device, close the water supply and Pull the plug out of the socket.

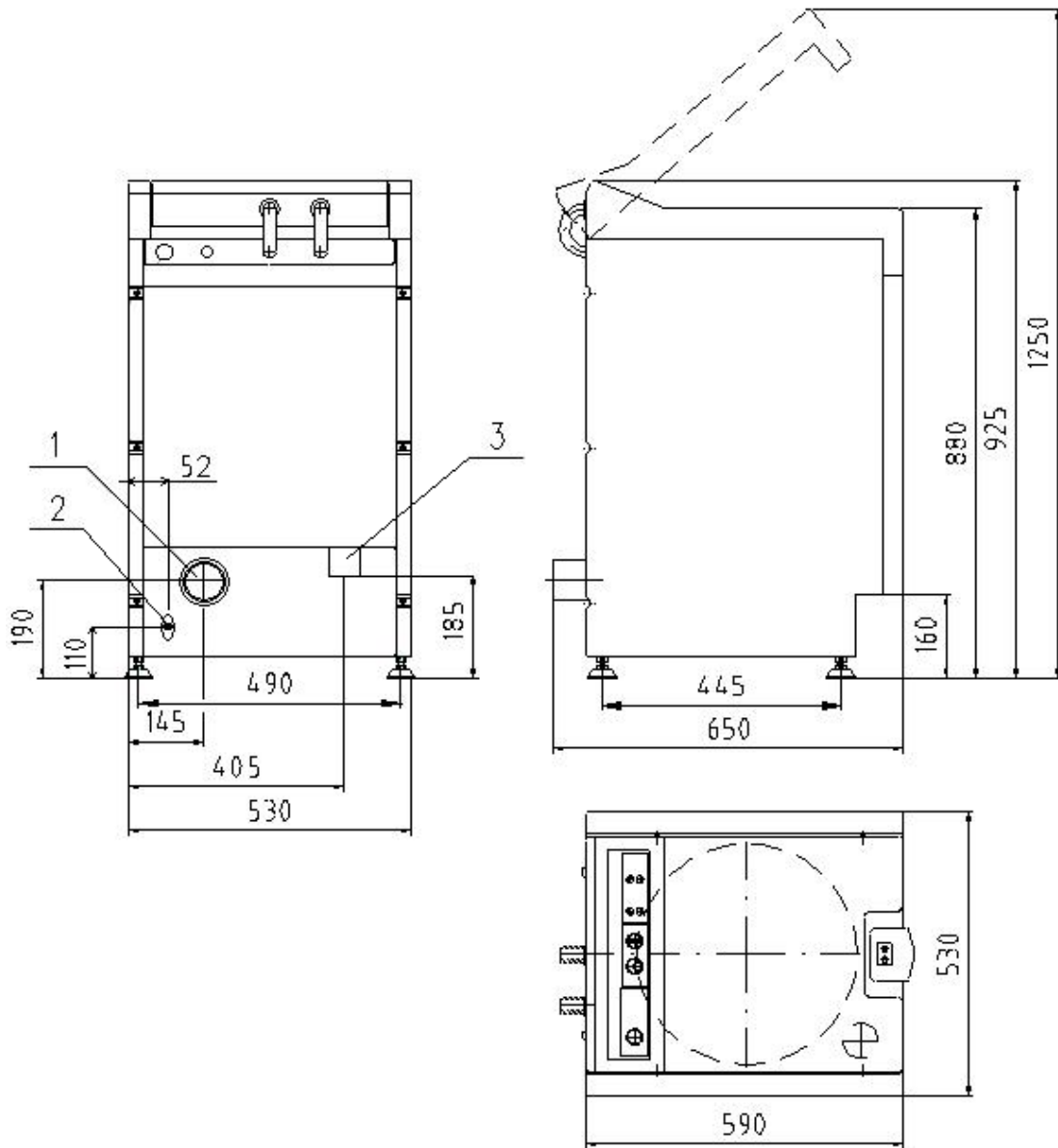
Maintenance of machines is possible in accordance with hygienic regulations. We recommend regular maintenance, at least once a month.

## Technical specification

### Technical specification

Water inlet – inlet valve	G 3/4", 0,1 - 0,8 Mpa
Connections to the toilet drain	DN 70
Load of paper containers	3 large paper containers
Duration of operation cycle	min 3 minutes max 19 minutes
Voltage / Frequency	220 V, 230 V, 50 Hz
Rated input power	2,4 kW
Power consumption per operation cycle	36 Wh
Power / motor	1,3 kW
Power / pump	0,25 kW
Electrical protection	16 A
Water consumption per cycle	30 liters
Volume of water tank	26 liters
Volume of disinfection container	2, 3, 4 or 5 liters
Drain connection - Diameter	70 mm
Weight	104 kg
Noise emission	69,3 dB
Dimensions w/h/d	53 / 85-93 / 65 cm
Class of protection	I
Degree of protection	IP X4
Connection	Power supply cord 3 x 1,5 mm <sup>2</sup> , length about 3 m
Rated speed of the motor	1350 Min <sup>-1</sup>
Design	stainless steel white coated


## Dimension drawing

1. Drain connection - ID 70mm
2. Water supply connection G 3/4" - L= 2m
3. Power supply connection 230V/50Hz L= 3m

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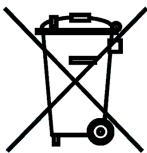At the end of the product's useful life, please dispose of it at appropriate collection points provided in your country.

**Attention**

This product has been manufactured and packaged in environmentally friendly aspects. We ask you to dispose of the packaging properly and use the washing machine according to the rules.

Art. Nr. 735342750002  
02/2020

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**BOSK GmbH